



Acquisition guide



TRANSMURAL BIOTECH S.L. C/BEETHOVEN 15 PLANTA 4 DESP 18. 08021. BARCELONA ESPAÑA



DATE: 05/07/2024

NUMBER OF EDITION: 1

Acquisition Guide for Osteoporosis




- **Anteroposterior (AP) radiograph (X-ray) of the hip in DICOM format.**
- Possibility of being on both sides (the entire pelvis) or only on the right or left (half pelvis).
- Images with a prosthesis on one of the two sides are accepted, although results will not be given for that side..



Axial or other views other than anteroposterior are not accepted.



The complete head of the femur(s) must be visualized.

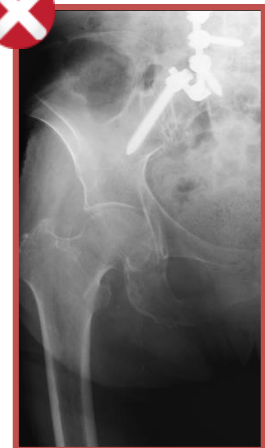

Nails or other elements should be avoided near the femur to be analyzed..

Clinical intake acquisition protocol:

Citation: Murphy A, Knipe H, Er A, et al. Pelvis (AP view). Reference article, Radiopaedia.org (Accessed on 25 Apr 2024) <https://doi.org/10.53347/rID-44863>

DOI: <https://doi.org/10.53347/rID-44863>

rID: 44863

Permalink: <https://radiopaedia.org/articles/44863>