



Acquisition Guide



Transmural Biotech S.L. ; Calle Juan Ignacio Luca de Tena 12, 28027 Madrid España



DATE: 10/06/2023

EDITION NUMBER: 1




- Image obtained with a retinograph.
- It is recommended not to use an image resolution less than 2 Mpx.
- The system accepts all sizes of images, but will rescale them to 1200x1200px. Therefore it is recommended to stick to this size as much as possible.




- Images must be in JPEG or PNG image format.
- The full retinal sphere should be seen occupying the entire image.
- Avoid borders or frames around the retina.
- The image must not be treated with any filter or processing of any kind.
- No measurement or marker should appear above the lesion.

The entire retina should be seen.

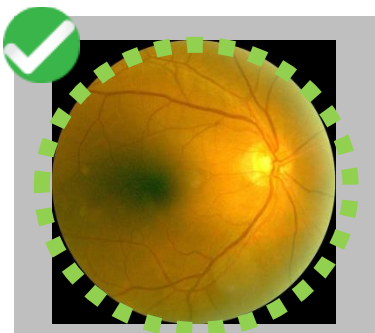

Avoid borders or frames.

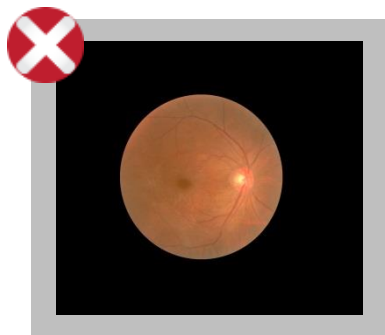

No measurement marker should appear.

